Mondocel[®] Nasal Tampon

INSTRUCTIONS FOR USE (EN)

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NAVODILA ZA UPORABO (SL)

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Instructions for use – ENGLISH

Contents

1. INTENDED USE
2. COMPOSITION
3. INDICATIONS FOR USE
4. CONTRAINDICATIONS
5. WARNINGS
6. PRECAUTIONS
7. PRODUCT DISPOSAL
8. PRINCIPLES OF OPERATION

1. INTENDED USE

Mondocel[®] Nasal tampons are intended to be placed temporarily, via a nostril, into the nasal cavity to control bleeding and edema by tamponade and excellent absorption capacity of the tampon.

The tampons are supplied as compressed devices to facilitate insertion into the nasal cavity and to ensure optimal positioning. These compressed devices rapidly expand with hydration and adapt well to the anatomical site.

Mondocel[®] Nasal tampons have a spongy, porous structure and are fiber-free.

2. COMPOSITION

Polyvinyl alcohol (PVA) foam

Mondocel[®] Nasal tampons are available in different shapes and sizes, with or without a string and airway tube.

3. INDICATIONS FOR USE

- Postoperative after nasal surgery
- Treatment of epistaxis

Mondocel[®] Nasal tampons should only be used by medical staff.

4. CONTRAINDICATIONS

The tubed products contain **silicone** which may cause an allergic reaction in patients with sensitivity to silicone.

5. WARNINGS

1) The use of internal tamponade devices can in rare cases and in particular conditions, perhaps following surgery, present a risk of **toxic shock syndrome (TSS)**. Toxic shock syndrome is a multisystem inflammatory

response due to infection by bacteria capable of producing certain endotoxins. Mondocel® Nasal tampons do not contain any antibiotics and exhibit no antimicrobial properties. They are not bacteriostatic towards pre-existing infection, nor do they prevent the occurrence of new infections. In the case of pre-existing infections it is recommended to institute adequate treatment.

2) In order to avoid dislocation and ingestion of the nasal tampon, it is recommended to fix the string of the nasal tampon on the patient's cheek using an adhesive plaster.

3) Mondocel® Nasal tampons must be kept hydrated during their stay in the nasal cavity and before their removal.

6. PRECAUTIONS

1) Do not use the tampons if package is breached or damaged.

2) Mondocel® Nasal tampons are supplied sterile; do not re-sterilize.

3) Do not apply alcoholic solutions on the tampon as this compromises the integrity, composition and functionality of the foam.

4) Mondocel® Nasal tampons are for single use only. Reuse of the device can lead to several hazards:

- cross contamination
- infection risks
- deterioration and alteration of the characteristics of the device due to the first use with consequent risk of injury to the patient

5) Do not use the tampons after the expiry date.

6) Keep the tampons away from sunlight and heat. Keep the tampons dry during storage and store them between 5°C and 35°C.

7. PRODUCT DISPOSAL

In order to minimize the risk of potential infection hazards or environmental pollution, Mondocel® Nasal tampons should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

8. PRINCIPLES OF OPERATION

Use clean or aseptic techniques for application, according to your institutional protocol.

Insertion

The tampon is inserted in dry, compressed state. To facilitate insertion, the tampon (from tip to halfway down) can be greased with a water or saline based lubricant.

The tampon is carefully inserted into the nasal cavity, either manually or with the help of a pair of tweezers, gently advancing antero-posterior and parallel to the nasal floor until the entire tampon is contained within the nasal cavity.

Sinus pack insertion: insert (lateral to the middle turbinate) up to the middle meatus.

If a nasal tampon with a string is used, the string must be attached to the cheek with a plaster (avoid knotting the string over the bridge of the nose) in order to prevent the tampon itself from mobilizing and moving backwards with the risk of being ingested or worse, inhaled.

When using a tampon with a ventilation tube, make sure that the entrance of the tube is never blocked.

Expansion

Once inserted, tilt the patient's head back and saturate the nasal tampon with sterile water or saline solution using a syringe (5 ml for tampons smaller than 45 mm and 10 ml for tampons larger than 45 mm).

This will not be necessary in case of epistaxis as the blood itself will cause the expansion.

The temperature of the sterile water or saline solution can affect the expansion rate. A cooler hydration solution results in a slower expansion rate, a warmer hydration solution results in a faster expansion rate.

The packing must kept moist while in use. Depending on the individual situation, the tampon must be hydrated one or more times a day, in order to prevent drying and consequent loss of elasticity of the nasal tampon.

Removal

Mondocel® Nasal tampons can be left in situ for maximum five days (post-surgery). For treatment of epistaxis it is recommended to remove the tampon earlier.

Before removal, the tampon will need to be hydrated with either sterile water or saline (wait five to ten minutes for proper hydration).

When removing a nasal tampon with ventilation tube, the tube should be removed first prior to carry out the hydration of the tampon through the resulting hole.

Use a pair of tweezers to grasp the tampon and gently remove it.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer, who will take care for reporting the applicable Competent Authority.

 Mondomed NV
Middenweg 20
BE-3930 Hamont-Achel

CE 1639

Phone: +32 11 440 990
Email: info@mondomed.be
<http://www.mondomed.be>

	European conformity		Do not resterilize
	Medical device		Do not use if package is damaged
	Manufacturer		Consult instructions for use
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
	Batch code		Temperature limit
	Catalogue number		Unique device identification
	Sterilized using irradiation		Green dot
	Double sterile barrier system		Dispose of packaging in an environmentally responsible way

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